



For returns policy and
warranty information
please visit:

www.daray.co.uk

Tel: 0844 375 9000 fax: 0333 321 0973
email: info@daray.co.uk url: www.daray.co.uk
Marquis Drive, Moira, Derbyshire, DE12 6EJ

QAM.HL560LED.0613.1



HL560
HEAD-LIGHT
OPERATING MANUAL

1. Introduction

Utilising LED technology, the HL560 is robust, lightweight and has no heat in the beam making it comfortable for the patient and user.

The head-light emits a near-daylight coloured beam ensuring tissue tones are accurately rendered.

With the benefit of being rechargeable and low power consumption the HL560 is an extremely cost effective head-light.

On/off button



Adjustable focus

2. Features

- Fully adjustable
- Focusable beam
- Tilting lens
- Rechargeable

3. To charge

- Remove the cover on the inside of the battery pack located on the headband.



- Place the battery inside the charger supplied and plug into a power socket



4. Operation

- To turn on, press the silver button on the light module.
- Press silver button again to dim the light
- Once dimmed press again for light to flash continuously.
- Finally press the button again to turn the light off.
- To adjust focus, either twist and turn or push and pull the light module.